**HEINE** QUALITY  
MADE IN GERMANY

## HEINE® HSL 150

**CE**

**HEINE Optotechnik GmbH & Co. KG**  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)81 52/38-0  
Fax +49(0)81 52/38-202  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
med 2314    2015-01-28

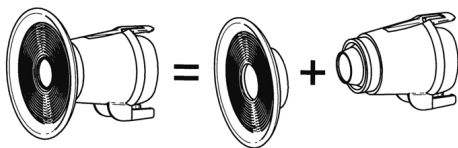
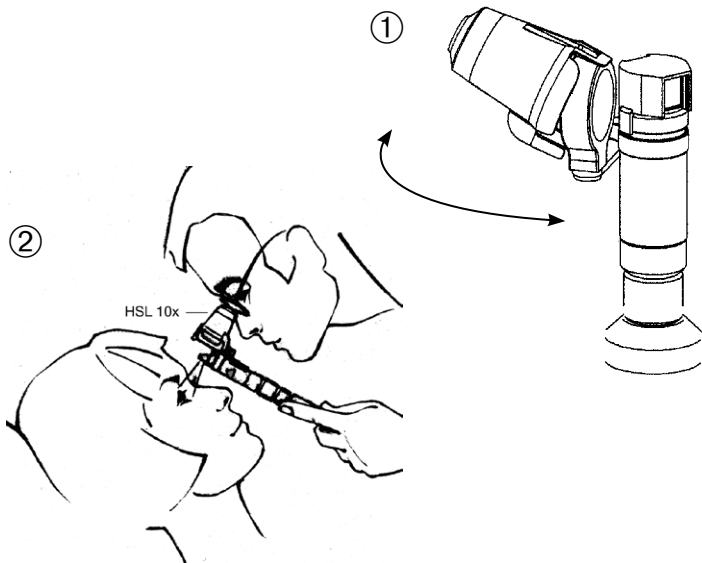
V-200.00.060

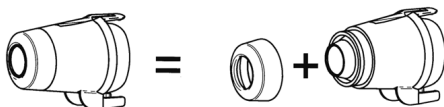
# HEINE® HSL 150

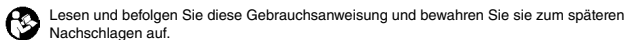
DEUTSCH	4
ENGLISH	7
FRANÇAIS	10
ESPAÑOL	13
ITALIANO	16
SVENSKA	19
NEDERLANDS	22
DANSK	25
SUOMI	28
PORTUGUÊS	31

③

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 Please read and follow these instructions for use of and keep them for future reference.

### Intended Use

HEINE® HSL 150 is a battery powered device. The devices contain illumination and viewing optics and are intended to examine the media, cornea, aqueous, lens, vitreous and the retina of the eye. They must be used only by qualified medical personnel. The instruments are intended for short-term examination; patients can be examined according to the specified exposure guidelines.

 **For U.S. only:**  
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

### Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

### Product overview

- 1 Loupe
- 2 Filter attachment
- 3 Slide control
- 4 Connector
- 5 HEINE HSL 10x loupe attachment
- 6 Bulb
- 7 Pin
- 8 HEINE Handle

### Setting up

To set up the instrument, plug the instrument head on to the HEINE handle (8).

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE XHL® 2.5V bulb  
only for use with the HEINE Battery handle

Red ring = HEINE XHL® 3.5V bulb  
only for use with the HEINE Rechargeable Handle

### Operation

Swivel the loupe (1) to the left to examine the right eye, and vice versa. Examine the patient's right eye with your right eye and vice versa.

Approach the patient from a 30° angle to a distance of about 4cm. Looking through the loupe, sweep the eye with the slit by turning the handle.

The image can be fine-focused by adjusting the distance from the HSL to the patient's eye.

The blue filter is selected by rotating the filter attachment (2) by 180°. Pull the attachment upwards to remove.

The slide control (3) can be used to adjust the size of the slit. The loupe provides magnification in the left and right working position.

With the loupe attachment HSL 10x (5), the magnification is increased by 10x. See figures ① ② ③ (Page 3).

HEINE® HSL 150 is intended for a brief examination of less than 2 minutes with a 15 minutes break until the next application.

The setup and operation of the HEINE handles are described in a separate instruction for use.

### Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: non-critical

Spaulding Classification USA: non-critical

 Allow the device to cool down before reprocessing.

In the event of suspected contamination, carry out hygienic preparation of the instrument.

The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the agents and procedures mentioned below. Other procedures/reprocessing agents have to be validated.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the disinfectant manufacturer.

Do not use spray or immersion disinfection, dripping wet or heavily foaming tissues. Do not use automated or ultrasonic reprocessing.

### Procedure

The HSL 150 can be cleaned and disinfected manually (wipe clean and wipe disinfect).

Recommended agents:

Cleaning agent: Neodisher® MediClean

Disinfectant agent: alcoholic (e.g. Incides® Tissues)

### Changing the light source

 Verify that the lamp voltage complies with the supply voltage of the handle.

Allow the device to cool down before changing the bulb.

Remove the HSL from HEINE handle, unscrew the connector (4) and pull off the bulb (6) carefully.

Wipe the glass of the new bulb with a soft, clean cloth and insert the bulb so that the pin (7) coincides with the groove.

Screw the connector (4) back in.

### Maintenance and Service

The instruments do not require maintenance or service.

### General Warnings

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

### General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

If you don't use the device for a longer period of time, please remove the batteries in advance.

 **Caution** – The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater the risk of ocular damage. Exposure to light from this instrument when operated at maximum intensity should not exceed 23 min (at a distance of 30 mm).

#### **Disposal**

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

#### **Electromagnetic Compatibility**

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

 This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.

The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission and reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices. If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains following tables:

- Guidance and manufacturer's declaration – Electromagnetic immunity
- Technical specification
- Explanation of the used symbols

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**Guidance and manufacturer's declaration – electromagnetic emissions**

The device is intended for use in the electromagnetic environment specified below.  
The customer or the user of the device should assure that it is used in such environment.

<b>Emission test</b>	<b>Compliance</b>	<b>Electromagnetic environment – Guidelines</b>
RF emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely that any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	Symmetrical three-phase devices and other devices.
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Passed	

**Guidance and manufacturer declaration - Electromagnetic immunity**

The device is intended for use in the electromagnetic environment specified below.  
The customer or the user of the device should assure that it is used in such environment.

<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – Guideline</b>
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/ burst IEC 61000-4-4	± 2 kV for mains cables ± 1 kV for input and output lines	± 2 kV for mains cables ± 1 kV for input and output lines	The supply voltage quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV voltage phase – phase, ± 2 kV voltage phase – earth	± 1 kV voltage phase – phase ± 2 kV voltage phase – earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% $U_T$ , (>95% dip in $U_T$ ) for 1/2 period 40% $U_T$ , (60% dip in $U_T$ ) for 5 periods 70% $U_T$ , (30% dip in $U_T$ ) for 25 periods <5% $U_T$ , (>95% dip in $U_T$ ) for 5 seconds	< 5% $U_T$ , (>95% dip in $U_T$ ) for 1/2 period 40% $U_T$ , (60% dip in $U_T$ ) for 5 periods 70% $U_T$ , (30% dip in $U_T$ ) for 25 periods <5% $U_T$ , (>95% dip in $U_T$ ) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered by a UPS (uninterruptible power supply) or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Comment:  $U_T$  is the a.c. supply voltage prior to application of the test level.

### Guidance and manufacturer's declaration – electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment Guidelines
Conducted RF IEC 61000-4-6	3 V <sub>eff</sub> 150 kHz to 80 MHz	3 V eff	Portable and mobile RF communication equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated HF IEC 61000-4-3	3 V/m 80MHz to 2,5GHz	3 V/m	<p><b>Recommended separation distance:</b>  <math>d = 3,5/3 \cdot \text{SQRT}(P/W)</math>  <math>d = 3,5/3 \cdot \text{SQRT}(P/W)</math> 80 MHz to 800 MHz  <math>d = 7/3 \cdot \text{SQRT}(P/W)</math> 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey<sup>a</sup>, should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note 1: At 80MHz and 800MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a** Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.
- b** Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V / m.

### Recommended separation distances for portable and mobile RF communication equipment and the device

The device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 3,5/3 \cdot \text{SQRT}(P)$	80 MHz to 800 MHz $d = 3,5/3 \cdot \text{SQRT}(P)$	800 MHz to 2,5 GHz $d = 7/3 \cdot \text{SQRT}(P)$
0.01	0.1	0.1	0.2
0.1	0.4	0.4	0.7
1	1.2	1.2	2.3
10	3.7	3.7	7.4
100	11.7	11.7	23.3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**Technical specification**

Environmental conditions for operation	+10°C to +35°C 30% to 75% rel. humidity 700hPa to 1060hPa
Environmental conditions for storage	+5°C to +45°C 45% to 80% rel. humidity 500hPa to 1060hPa
Environmental conditions for transport	-20°C to +50°C 45% to 80% rel. humidity 500hPa to 1060hPa
Nominal voltage	3.0V – 3.7V
Nominal current Typ	440–760 mA
Protection class	internal power supply
Device classification according ISO 15004-2	Group 2

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### Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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